



Alternate Step 5: Connecting the CPAP Interface

Insert the Zmachine CPAP Interface between the mask and CPAP machine air hoses.

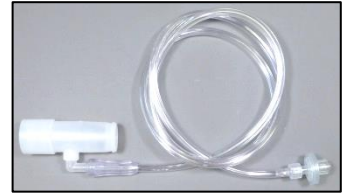

1. Push both hose ends tightly into the pressure adapter as shown.

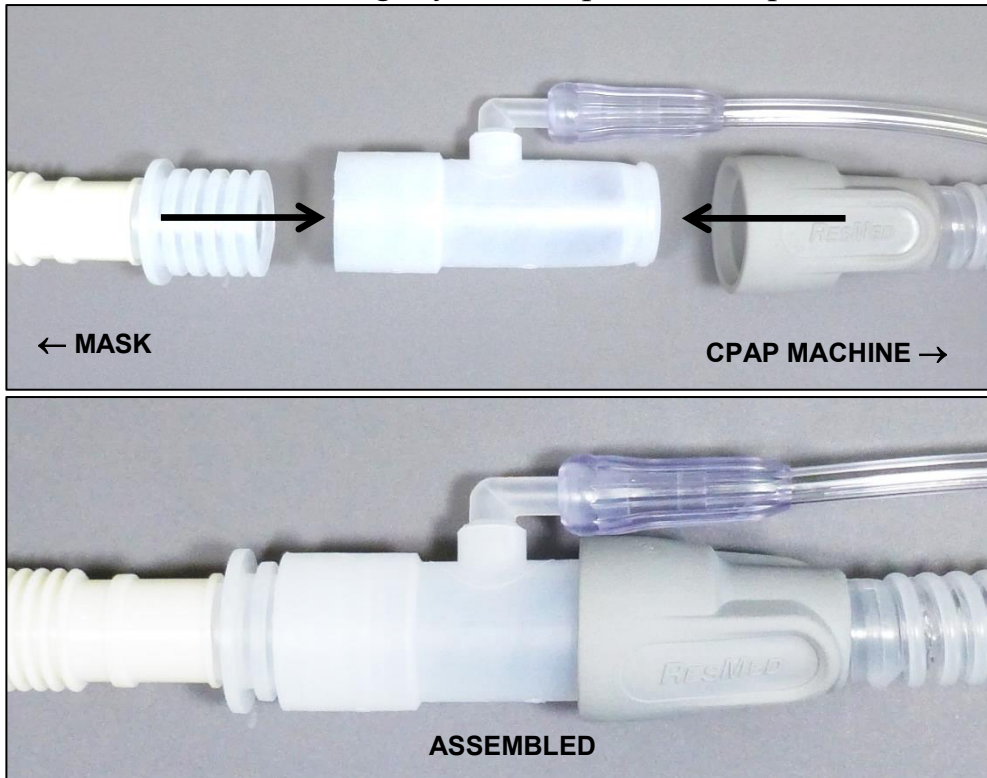

Ensure that all connections are tight and secure.

2. Twist the free end of the CPAP Interface clockwise onto the Zmachine fitting as shown.



Do not overtighten
Tightening beyond snug may damage the device.

When your test is complete...

1. Disconnect the Zmachine CPAP Interface and discard.
2. Reconnect your mask to your CPAP machine.